

# **Urgent Field Safety Notice**

Exactech Equinoxe Reverse Shoulder Fixed Angle Torque Defining Screw Kit

FSCA-identifier: 11012016 Type of action: Field Advisory

Date: October 17, 2016

Attention: All staff who may use or handle these devices

# Description of Issue:

When the Torque Defining Screw is tightened to the torque required to disengage the screw head, the screw head will disengage as expected, but it may not be retained in the Torque Defining Screwdriver.

## Clinical Impact (Risk to Health):

This may cause a delay in surgery of less than 1 minute due to the need to pick up the screw head that was not retained by the driver. Surgeons should also be advised that there is a potential for the screw head to fall into the wound site.

#### Details of the affected devices:

Catalog Number	Serial Ranges	Device Description
320-20-00	4410696-4410819	Equinoxe Reverse Shoulder Fixed Angle Torque Defining Screw Kit
	4415393-4415492	
	4418789-4418888	
	4422658-4423007	
	4423542-4423591	
	4424509-4424558	
	4458153-4458252	
	4458303-4458352	
	4461732-4461806	
	4464592-4464691	
	4467733-4467824	
	4471099-4471298	
	4485494-4485543	
	4488224-4488260	
	4504738-4504987	
	4513150-4513199	
	4513250-4513299	
	4517116-4517215	
	4519806-4520005	
	4533291-4533390	
	4542043-4542092	



352-377-1140 FAX 352-378-2617

# Advice on action to be taken by the user:

- Extend this information to your accounts that may have this product in their possession.
- Acknowledge receipt of this notice by completing and returning the attached form.

# Transmission of this Field Safety Notice:

This notice must be forwarded to all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred.

## Contact reference person:

Tina O'Brien

#### National Distributor for Exactech affected devices:

Exactech UK Grosvenor House Prospect Hill Redditch B97 4DL United Kingdom

Thank you for your prompt attention to this matter. Though we do not believe that this issue has an impact on patient safety at this time, we are taking this action to ensure the health and safety of patients and the users of our products. Corrective actions of this type are collaborative efforts and require your participation to be effective. We regret any inconvenience that this action may cause, but we appreciate your understanding as we take action to ensure patient and customer satisfaction.

The undersigned confirms that this notice has been notified to the appropriate Regulatory Agency.



Exactech, Inc.

Attachment: Response form



352-377-1140 FAX 352-378-2617

# RESPONSE FORM - FIELD ADVISORY NOTICE

have received this notice and will extend it to my accounts that may have the affected Exactech quinoxe Reverse Shoulder Fixed Angle Torque Defining Screw Kit (REF 320-20-00).
vistributorship
fame and Title (Print)
Jame (Signature)
hank you in advance for your prompt attention to this matter. According to policy, please return his completed response form via fax or email within 48 hrs of receipt of the notice and contact our Exactech representative to confirm quantities at your location.
xactech representative
Tina O'Brien
Exactech UK
Grosvenor House

Prospect Hill Redditch B97 4DL

email: tina.obrien@exac.co.uk