COOK	Quality System Form						
	Document Number:		Revision:	QMS Owner:	Page:		
	D00060366		011	Cook Medical Europe Ltd.	1 of 2		
	Title:	Customer Comm	nunication				
Lana ava Munaham		E44 00E					

Legacy Number: | F14-00E

# COOK

Cook Medical Europe

O'Halloran Road, National Technological Park, Limerick, Ireland. Phone: + 353 61 334440 Fax: + 353 61 334441

## **Customer Communication**

Commercial name of the affected product: Hemospray Endoscopic Hemostat Manufacturer: Wilson-Cook Medical Cook Reference Number: 2018FA0011

Date: 10 December 2018

Attention: Chief Executive/Risk Management/Purchasing

### Details on affected devices:

Hemospray Endoscopic Hemostat

PRODUCT BRAND NAME	Catalog Identifier	
	HEMO-7-EU	
Hemospray Endoscopic Hemostat	HEMO-10-EU	

### Description of the problem:

Cook Medical has become aware that the Hemospray Endoscopic Hemostat has been used to treat patients with upper gastrointestinal bleeding that is accessed for treatment by using an endoscope in the retroflexed position. We have received reports of the Hemospray Endoscopic Hemostat powder adhering to the endoscope shaft after being sprayed in the retroflexed position, which led to difficulty or inability removing the endoscope from the patient.

Cook Medical has revised the Hemospray Endoscopic Hemostat Instructions for Use (IFU) to include a statement to bring awareness to the potential for this clinical situation that states "When spraying in retroflexed position, Hemospray powder may adhere to the outside of the endoscope. This may result in difficulty repositioning/removing the endoscope, particularly if passing through a strictured area." Please see Potential Complications section of the attached copy of the IFU for further information.

### Advise on action to be taken by the user:

No devices need to be returned. Please do the following:

- Please share this notice with others in your organization who use this device so they are aware of this clinical situation.
- Maintain a copy of this notice for your records.
- Complete and return via email or facsimile the attached Customer Response Form by e-mail to <u>European.FieldAction@cookmedical.com</u> or by fax marked for the attention of European Customer Quality Assurance to + 353 61 239294.
- Please report any adverse events to Cook Medical by contacting our Customer Support Department. (e-mail <u>SSCProduct.Complaints@CookMedical.com</u>, phone +353 61 239252).
- Please contact your Cook Sales Representative if further information is desired.

	Quality	Quality System Form							
COOK	Document Number:		Revision:	QMS Owner:	Page:				
MEDICAL	D00060366		011	Cook Medical Europe Ltd.	2 of 2				
	Title:	Customer Communication							
Legacy Number:		F14-00E							

### Transmission of this Customer Communication: (if appropriate)

This notice must be shared with appropriate personnel, including down to the user level, within your organization or with any organization where the potentially affected devices have been transferred. Thank you for your immediate attention to this matter. Should you have any questions, please feel free to contact us for more information (e-mail: <u>European.FieldAction@CookMedical.com</u>, phone +353 61 334440).

Contact reference person:



+1(336)740-0157

WARNING CONFIDENTIAL PROPRIETARY PROPERTY - This document is owned by COOK Medical. It contains confidential proprietary trade secret information and must not be copied. The document and the information it contains can be used only by the recipient for the specific use for which it was requested. All other use is strictly prohibited. This document must be returned to COOK Medical immediately upon request by COOK Medical. By possession of this document, the possessor expressly agrees to comply with these terms. "© COPYRIGHT Cook Medical Europe Ltd. 2017"