

Teleflex Medical IDA Business & Technology Park Dublin Road, Athlone Westmeath, Ireland

15 March 2019

#### **URGENT - FIELD SAFETY NOTICE**

Type of Action	Recall
Teleflex Reference	EIF-000353
Commercial Name	Gibeck HME Filter + Catheter Mount RUSCH Catheter Mount RUSCH Connectors Bulk for Breathing Circuits RUSCH Connectors for Breathing Circuits
Product Code/Lot Number	Refer to Appendix 4

Dear Customer,

#### **Details of affected devices**

Teleflex has initiated a voluntary Field Safety Corrective Action (FSCA) for the above listed product, refer to Appendix 4 for a list of product codes and lots impacted.

#### Description of the problem & immediate actions required

Teleflex Medical is initiating a Field Safety Corrective Action for the above-mentioned products because there is a possibility for the swivel and cobb connectors to detach during use. If detachment occurs prior to use, it should be readily identified by the user during set-up. If detachment occurs during use and presents as a leak in the breathing circuit, ventilator pressure alarms will alert, and intervention could be required to replace the connector with another catheter mount or else reconnect the breathing circuit directly to the Endotracheal tube until another new catheter mount is available. If there is a delay in replacing the catheter mount or a delay in reconnecting the breathing circuit, the leak may lead to desaturation until the catheter mount is replaced and normal ventilation is maintained.

Our records indicate that you have received product that is subject to this Field Safety Corrective Action. We are now notifying our customers to take the following actions.

#### Important note, depending on your device location please adhere to the following Action list only:

Device location	Action List Number
Home setting	1
Medical facilities (hospitals, medical staff, etc.)	2
Distributors	3



# Action list number 1 Home setting

- 1. We request that you check your inventory for product within the scope of this FSCA, refer to Appendix 4 for the list of impacted codes & lots. Users should **cease use** of impacted product and quarantine immediately. For support, contact your customer service representative.
- **2.** If product listed in Appendix 4 is already in use, Teleflex recommend replacing the catheter mount. Refer to Appendix 3 for a list of Teleflex substitute product codes.
- **3.** If you do not have stock as listed in Appendix 4, mark the according checkbox on the Acknowledgement Form (Appendix 1) and return the form to the fax number or e-Mail-address mentioned below.
- **4.** If you do have stock as listed in Appendix 4, mark the according checkbox on the Acknowledgement Form (Appendix 1) and contact customer service by calling the phone number mentioned below. Customer Service will issue you with a return number. Write the return number into the respective field in the Acknowledgement Form and return this form immediately to Customer Service.
- **5.** Teleflex (or your local dealer) will issue a credit note upon receipt of the returned affected product.
- 6. Should you require further information or support concerning sourcing alternative product, please contact your customer service representative.

# Action list number 2 Medical facilities

- 1. We request that you check your inventory for product within the scope of this FSCA. Users should cease use and distribution of impacted product and quarantine immediately. Medical facilities have two options
  - **a.** You may then return all product in scope, refer to Appendix 4 for the list of impacted codes & lots, to Teleflex
  - **b.** Alternatively, you may inspect and return only product that is defective. Refer to Appendix 2 for detail on inspection instructions.
- **2.** If product in scope is in situ, Teleflex recommend replacing the catheter mount. Refer to Appendix 3 for a list of Teleflex substitute product codes.
- **3.** If you do not have stock in scope of this FSCA mark the according checkbox on the Acknowledgement Form (Appendix 1) and return the form to the fax number or e-Mail-address mentioned below.
- **4.** If you do have stock in scope of this FSCA, mark the according checkbox on the Acknowledgement Form (Appendix 1) and contact customer service by calling the phone number mentioned below. Customer Service will issue you with a return number. Write the return number into the respective field in the Acknowledgement Form and return this form immediately to Customer Service.
- 5. Teleflex (or your local dealer) will issue a credit note upon receipt of the returned affected product.



### Action list number 3 Distributors

- 1. Provide this field safety notice to all customers who have received product in scope of this FSCA. Your customer is then required to complete the acknowledgement form and return to you.
- **2.** Refer to Appendix 3 for a list of Teleflex alternative product, please call Teleflex customer service should you require assistance regarding same.
- **3.** We request that you check your inventory for product within the scope of this FSCA. Cease use and distribution of impacted product and quarantine immediately. Distributors have two options
  - **a.** You may then return all product in scope, refer to Appendix 4 for the list of impacted codes & lots, to Teleflex
  - **b.** Alternatively, you may inspect and return only product that is defective. Refer to Appendix 2 for detail on inspection instructions.
- **4.** As a distributor, you are required to confirm to Teleflex that you have completed the field activity outlined above. Upon completion of your actions, please forward the completed Acknowledgement Form to Customer Service.
- **5.** Please be aware that all European Economic Area/Switzerland (EEA/CH) and Turkey Member State Competent Authorities in which Teleflex distribute directly will be notified by Teleflex.
- **6.** If you have further distributed product outside of your country, please notify Teleflex by return email to the email address below.
- 7. If you are a distributor and/or have a reporting responsibility within or outside the EEA/CH/TR region, please notify your local Competent Authority of this action. Please forward the notification and all communication with your local competent authority to Teleflex.

#### **Teleflex**

Teleflex informs all customers, employees of Teleflex and distributors of this Field Safety Corrective Action.

#### **Transmission of this Field Safety Notice**

This notice should be passed on to all persons who need to be aware within your organization or to any organization where the potentially affected devices have been transferred. Please consider end users, clinicians, risk managers, supply chain/distribution centres etc. in the circulation of this notice. Maintain awareness of this notice until all required actions have been completed in your organisation.

#### **Contact reference person**

Should you require any further information or support concerning this issue, please contact:

Customer Service:Produktmanagement:Kontakt: Customer ServiceKontakt: Nadja Sommerfeld

Fax: 0711 / 49 05 06 08

**E-Mail:** recalls.de@teleflex.com

Please be advised that all European Economic Area/Switzerland (EEA/CH) and Turkey Member State Competent Authorities to which Teleflex distribute directly will be notified by Teleflex. Teleflex is committed to providing high quality, safe and effective products. We sincerely apologize for any inconvenience this action may cause your operations. If you have any other questions, feel free to contact your local sales representative or Customer Service.

For and on behalf of Teleflex,



Appendix 1

Customer No.

# FIELD SAFETY CORRECTIVE ACTION ACKNOWLEDGEMENT FORM

#### **IMMEDIATE ATTENTION REQUIRED**

Ref. EIF-000353

#### **RETURN COMPLETED FORM IMMEDIATELY TO:**

FAX: +353 (0) 1 4370773

Email: Recalls.Intl@teleflex.com

We confirm receipt of this FSN and have completed the required actions contained therein. We confirm our inventory DOES include products affected by this FSCA. The use and further distribution of impacted product has stopped. All impacted product is on hold and the amount below will be returned.

Return Authorisation No.

PLEASE PRINT PRODUCT QUANTITY NUMBERS CLEARLY

PRODUCT NUMBER	LOT NUMBER	QUANTITY (Returning)

- Include a copy of the completed Acknowledgement Form in the returns package with the returned units
- Ensure the RAN number is clearly visible on the returns package
- Please label returns as "Field Action Returns"

Complete this Acknowledgement form and return immediately by using the fax number or e-mail address above.

INSTITUTION NAME (EG NAME OF HOSPITAL, HEALTH CARE ORGANISATION)					
INSTITUTION ADDRESS	Phone / Fax				
FORM COMPLETED BY:	Stamp				
PRINT NAME:					
SIGNATURE:					
DATE					



#### **INSPECTION INSTRUCTIONS**

#### **Applicable to Medical Facilities and Distributors only**

Product in scope of this Field Safety Corrective Action (FSCA) contains identification numbers (ID's) embossed on the cobb connector. While Teleflex could not refine the scope of this action via lot numbers, product may be segregated by the ID number into conforming and non-conforming. This is an optional inspection step for Medical Facilities and Distributors only. Should you wish to return all impacted codes and lots without performing this inspection you may do so.

#### **Instructions:**

- 1. If product is still in the packaging (i.e. not in situ) it may need to be manoeuvred within the package so that the ID number is clearly visible. See Figure 1 and Figure 2 below for the location of the ID number
- 2. Products should be visually inspected to identify the ID embossed on the product
- **3.** If the ID number on the product is listed under table 1 below, the product is potentially non-conforming product and should be returned to Teleflex
- **4.** If the ID number is not listed under table 1 below, those units are conforming, and you may retain that product for use/distribution

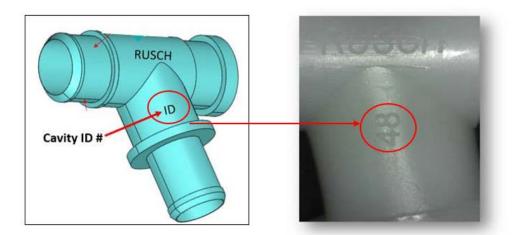


Figure 1 – Cobb Connector

Figure 2 – Identification number (ID)

ID Numbers	
Non-conforming Product	
39	If the ID number embossed on the catheter mount is 39, 42, 45 or 48 –
42	Return to Teleflex
45	netalli to relenex
48	

Table 1



	Substitute Teleflex Product Codes & Comments							
Product Code in scope of Field Action	Substitute 1	Substitute 2	Substitute 1 - Comment	Substitute 2 - Comment				
191010-000100	191009-000100	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191010-000150	191009-000150	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191010-000200	191019-000200	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191010-T00150	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
		Ţ						
191011-000050	191009-000050	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191011-000100	191009-000100	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191011-000150	191009-000150	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191011-000200	191019-000200	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191011-000300	191020-000300 (only in 22F)	22531	straight connector	14-20cm with double Swivel but only with 22F to machine side				
	_							
191013-000150	191009-000150	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191013-000200	191019-000200	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191013-000250	191020-000300	22531	straight connector in 30cm	14-20cm with double Swivel but only with 22F to machine side				
191014-000100	191025-000100	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191014-000150	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191014-000200	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191014-T00150	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191015-000050	191021-000050	22531	straight connector	14-20cm with double Swivel but only with 22F to machine side				
191015-000100	191025-000100	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191015-000150	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191015-000200	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				



	Substitute Teleflex Product Codes & Comments							
Product Code in scope of Field Action	Substitute 1	Substitute 1 Substitute 2 Substi		Substitute 2 - Comment				
191015-T00150	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191033-000000	191009-000100	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191034-000000	191009-000100	22531	fixed elbow	14-20cm with double Swivel but only with 22F to machine side				
191036-000000	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191037-000000	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191525-000000	1624	-	Elbow connector w/o Swivel	-				
191526-000000	1624	-	Elbow connector w/o Swivel	-				
191527-000000	1624	-	Elbow connector w/o Swivel	-				
191569-000000	1624	-	Elbow connector w/o Swivel	-				
191657-000150	19092	-	fixed elbow & Filter	_				
191658-000100	19092	-	fixed elbow & Filter	T-				
191658-000150	19092	-	fixed elbow & Filter	-				
191659-000150	19092	-	fixed elbow & Filter	-				
191727-000150	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
191727-000200	22531	-	14-20cm with double Swivel but only with 22F to machine side	-				
	1							
191771-518150	19092	-	fixed elbow & Filter	-				

Product Name	Product Code	Lot/Batch Number	Product Name	Product Code	Lot/Batch Number
Gibeck HME Filter + Catheter Mount	191657-000150	201745	RUSCH Catheter Mount	191015-000100	18BT13
		201749			18CT03
		201802			18CT12
		201811			18CT19
		201816			18CT27
		201818			18CT32
		201821			18DT04
		201828			18DT29
		201831			18ET21
		201832			18ET36
		201836			18FT03
		201841			18FT07
		201842			18FT30
		201852			18FT40
	191658-000100	17KT09	7		18FT45
		17KT20			18GT07
		17KT24			18GT15
		17LT18			18GT21
		17LT19			18GT30
		18AT29			18HT14
		18DT24			18HT21
		18FT27			18HT26
		18FT42			18HT34
		18FT44			18IT05
		18GT14			18IT24
		18GT28			18JG07
		18HT02			18JG28
		18HT16			18JG29
		18HT20			18JT05
		18IT01			18JT11
		18IT14			18KG06
		18IT19			18KG20
		18IT27			18KG31
		18JT14			18KG32
		18KT02			18KG42
		18KT10			18KT06
		18LT12			18LG10
		18LT17			18LG20
		19AT01			18LG32
		19AT03			18LG35
	191658-000150	17KT08			18LG36
		17KT15			18LT21
		17LT07			19AG06
		17LT11		191015-000150	201826
		17LT21			17KT03
		18DT24			17KT10
		18FT37			17KT17
		18GT24			17KT25
		18KT02			17LT05
		18KT24			17LT12
		18LT08			17LT22
		18LT26			17LT31
	191659-000150	17LT10			18AT05
		18BT04			18AT10
		18FT36			18AT11
		18IT07			18AT18
	191771-518150	201820	7		18AT27
		201822			18BG09
		201825			18BT02
		201826			18BT07
		201828			18BT13
		201836			18BT20
		201837			18CT03
		201837			18CT12
		201901			18CT19
		201901			18CT27
	191010-000100	201902	†		18CT32
RUSCH Catheter Mount					
RUSCH Catheter Mount	191010-000150	201745			18DT13

Product Name	Product Code	Lot/Batch Number	Product Name	Product Code	Lot/Batch Number
RUSCH Catheter Mount	191010-000150	201802	RUSCH Catheter Mount	191015-000150	18DT17
		201805			18DT21
		201806			18DT29
		201808			18DT30
		201809			18ET04
		201811			18ET14
		201812			18ET21
		201816			18ET27
		201828			18ET36
		201831			18FT03
		201832			18FT19
		201837			18FT23
		201839			18FT25
		201842			18FT30
		201844			18FT40
		201846			18FT45
	191010-000200	201803	-		18FT49
	191010-T00150	201805			18GT15
		201810			18GT21
	101011 000050	201842 17KT10	-		18GT30
	191011-000050	17KT10			18HT06
		17KT17			18HT14 18HT21
		17KT25 17LT05			18HT26
		17LT12			18HT34
		17LT22			18IT05
	:	18AT18			18IT11
		18BT07			18IT17
		18BT20			18IT24
		18CT03			18JG28
		18DT13			18JT05
		18DT29			18JT11
		18ET37			18KG06
		18FT25			18KG12
		18GT07			18KG20
		18GT15			18KG31
		18GT21			18KG32
		18HT06			18KG42
		18HT14			18LG20
		18HT21			18LG32
		18IT05			18LG36
		18IT17			19AG06
		18LG32			19AG16
		19AG16		191015-000200	17KT03
	191011-000100	17KT10			17KT10
		17KT17			17KT17
		17KT25			17KT25
		17LT05			17LT05
		17LT12			17LT12
		17LT22 17LT31			17LT38 18AT11
		17LT31 18AT11			18AT18
		18AT18		ì	18AT27
		18BG09		}	18BG09
		18BT07		}	18BT02
		18BT13		•	18BT07
		18CT12			18CT27
		18CT12			18DT13
		18CT32			18DT21
		18DT04			18DT29
		18DT13			18ET04
		18DT29			18ET14
		18ET21			18ET21
		18ET36			18ET27
		18FT19			18ET36
		18FT25			18FT03
		18FT45			18FT30
		18GT04			18FT40
			•		

RUSCH Catheter Mount	Product Code 191011-000100	Lot/Batch Number  18GT15  18GT21  18GT30  18HT06  18HT14  18HT21  18HT26  18HT34  18IT05  18IT17  18IT24  18JG28  18JG29  18JT15  18JT11	RUSCH Catheter Mount	191015-000200 191015-T00150	Lot/Batch Number  18GT04  18HT26  18HT34  18IT11  18IT17  18JG29  18KG06  18KG12  19AG16  19AG28  17KT10
		18GT21 18GT30 18HT06 18HT14 18HT21 18HT26 18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05			18HT26 18HT34 18IT11 18IT17 18JG29 18KG06 18KG12 19AG16 19AG28
		18GT30 18HT06 18HT14 18HT21 18HT26 18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	18HT34 18IT11 18IT17 18JG29 18KG06 18KG12 19AG16 19AG28
		18HT06 18HT14 18HT21 18HT26 18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	18IT11 18IT17 18JG29 18KG06 18KG12 19AG16 19AG28
		18HT14 18HT21 18HT26 18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	18IT17 18JG29 18KG06 18KG12 19AG16 19AG28
		18HT21 18HT26 18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	18JG29 18KG06 18KG12 19AG16 19AG28
		18HT26 18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	18KG06 18KG12 19AG16 19AG28
		18HT34 18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	18KG12 19AG16 19AG28
		18IT05 18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	19AG16 19AG28
		18IT17 18IT24 18JG28 18JG29 18JT05		191015-T00150	19AG28
		18JG28 18JG29 18JT05		191015-T00150	
		18JG28 18JG29 18JT05		191015-T00150	17KT10
		18JG29 18JT05			
		18JT05			17KT17
		18JT05			17LT12
		181111			18AT18
		18KG06			18BT07
		18KG12			18DT13
					18FT25
		18KG20			18FT45
		18KG42			18HT21
		18LG10			18HT34
		18LG35			18IT11
		18LG36			18IT24
		18LT21			18KG31
		19AG06			
		19AG16			18KG32
		137,1010			18LG10
	191011-000150	17KT03			18LG20
		17KT10			18LG35
		17KT17			19AG06
		17KT25			19AG16
					19AT06
		17LT05			
		17LT12		191033-000000	17KT03
		17LT22			17KT10
		18AT05			17KT25
		18AT27			17LT05
		18BG09			17LT22
		18BT02			17LT31
		18BT07			17LT38
		18BT13			18AT05
		18CT12			18BT02
		18CT19			
		18CT27			18BT07
					18BT13
		18CT32			18BT20
		18DT13			18CT03
		18ET04			18CT12
		18ET14			18CT19
		18ET36			18CT27
		18ET37			18ET14
		18FT19			18ET21
		18FT25			18ET36
		18FT30			18FT19
		18FT45			18FT30
		18GT04			
		18GT07			18FT40
		18GT07 18GT15			18FT45
					18GT04
		18GT21			18GT07
		18GT28			18GT15
		18GT30			18GT21
		18HT06			18GT30
		18HT14			18HT06
		18HT21			18HT14
		18HT26			18HT21
		18HT34			18HT26
		18IT05			18IT05
		18IT17			
					18IT24
		18IT24			18JG28
		18JG28			18JT05
		18JG29			18KG12
		18JT05			18LG20
		18JT11			18LG36
					19AG06

Product Name	Product Code	Lot/Batch Number	Product Name	Product Code	Lot/Batch Numbe
RUSCH Catheter Mount	191011-000150	18KG06	RUSCH Catheter Mount	191034-000000	201744
		18KG12			201750
		18KG31			201751
		18KG32			201803
		18LG10			201805
		18LG32			201808
		18LG36			201819
		19AG06			201820
	101011 00000				201822
	191011-000200	17KT03			201825
		17KT17			201826
		17LT05			201828
		17LT12			
		18AT11			201829
		18AT18			201832
		18AT27			201839
		18BT07			201840
		18CT12			201842
		18CT19			201844
		18DT04			201846
		18DT13		101036 000000	201744
		18DT29		191036-000000	201744
		18ET37			201745
		18FT03			201750
		18FT07			201751
		18FT19			201752
		18GT07			201801
					201806
		18GT15			201807
		18GT21			201808
		18JG29			201809
		18JT05			201810
		18KG06			201812
		18KG12			201817
		18KG31			201818
		18LG10			201819
					201820
	191011-000300	17KT25			201821
		17LT22			
		18AT05			201822
		18AT27			201823
		18BG09			201826
		18DT14			201827
		18DT17			201828
		18FT07			201829
		18HT14			201830
		18IT17			201831
					201832
		18IT24			201834
		18JG28			201836
		18JT11			201840
		18KG06			201848
		18KG31			201850
	191013-000150	18HT26	1	191037-000000	17KT03
		18JG28			17KT10
	191013-000200	18AT27			
	121012-000500				17KT25
		18BT13			17LT05
		18ET04			17LT22
		18FT03			17LT31
		18GT04			18BT02
		18GT07			18BT07
		18GT30			18BT13
		18JG28			18CT12
		18JT11			18CT32
		18KG31			18ET21
	191013-000250	18BG09	1		
	121012-000520				18ET36
		18CT27			18FT03
		18FT03			18FT19
		18GT04			18FT25
		18HT06			18FT30
		18KG31			18FT45
	191014-000100	201746			18GT04
	:	. · · ·		i	

Product Name	Product Code	Lot/Batch Number		Product Code	Lot/Batch Number
RUSCH Catheter Mount	191014-000100	201748	RUSCH Catheter Mount	191037-000000	18HT26
		201750			18IT11
		201806			18IT24
		201807			18JG07
		201808			18JG28
		201810			18JG29
		201816			18JT11
		201820			18KG12
		201822			18KG20
		201825			18KG42
		201828			18LG10
		201834			18LG20
		201836			18LG32
					18LG35
		201839			
		201841			18LG36
		201843		404707 000470	19AG16
		201844		191727-000150	201744
		201846			201745
		201848			201747
	191014-000150	201747			201748
		201748			201749
		201749			201801
		201752			201802
		201801			201803
		201802			201805
		201803			201810
		201804			201811
		201805			201814
		201806			201815
		201810			201816
		201811			201818
		201814			201820
		201815			201821
		201816			201822
		201817			201825
		201818			201826
		201819			201827
		201820			201828
		201821			201829
		201822			201830
		201825			201831
		201826			201832
		201827			201834
		201829			201835
		201830			201836
		201831			201838
		201834			201839
		201835			201841
		201836			201842
		201837			201844
		201837			201846
		201842			201848
		201843			201851
		201845			201851
		201846		191727-000200	201745
		201847			201743
		201848			201747
	191014-000200	201745	1		201802
	131014-000200	201745			201810
		201747			201811
		201802			201815
		201805			201820
		201806			201821
		201810			201823
		201811			201826
		201815	DUCCH Comported Built for Built Co.	101500 000000	201828
		201816	RUSCH Connectors Bulk for Breathing Circuits	191569-000000	201841
		201817	RUSCH Connectors for Breathing Circuits	191525-000000	201743
		201822		i .	201752

#### EIF-000353 - Appendix 4

Product Name	Product Code	Lot/Batch Number	Product Name	Product Code	Lot/Batch Number
RUSCH Catheter Mount	191014-000200	201823	RUSCH Connectors for Breathing Circuits	191525-000000	201815
		201825			201816
		201828			201818
		201847			201819
		201848			201820
		201849			201821
		201850			201823
		201852			201826
		201902			201827
	191014-T00150	201745			201828
		201746			201833
		201747			201840
		201748			201844
		201749		191526-000000	17KT17
		201801		;	17LT05
		201811			17LT12
		201815			17LT22
		201825			17LT38
		201826			18AT05
		201827			18AT11
		201828			18BT02
		201829			18CT12
		201830			18DT29
		201831			18ET04
		201834			18ET21
		201836			18ET27
		201837			18FT30
		201840			18FT49
		201847			18GT15
		201851			18GT21
		201852			18GT30
	191015-000050	18HT06	1		18HT14
	131013 000030	18HT34			18JG29
		18IT11			18JT11
	191015-000100	17KT10			18KG12
	151015-000100	17KT17		·	18KG20
		17KT25			18KG31
		17LT12			18KG32
		17LT22			18LG10
		17LT31			18LG20
		18AT12		101537 000000	19AG16
		18AT18		191527-000000	201745
		18AT27			201746
		18BG09			201747
		18BT02			201751
		18BT07		1	201752