

January 16th, 2020 Olympus reference: QIL 152-008

URGENT FIELD SAFETY NOTICE

NOTIFICATION OF INCOMPATIBLE ENDO-THERAPY ACCESSORIES FOR OLYMPUS EVIS LUCERA PLEURAVIDEOSCOPE LTF-260

Attention: Operating Room Manager, Risk Management Department

	Model Name	Serial number
EVIS LUCERA PLEURAVIDEOSCOPE	LTF-260	All

Dear Health Care Practitioner,

Olympus has become aware of an issue that requires your attention. This Field Safety Notice pertains to the Olympus Pleuoravideoscopes LTF-260. The Pleuoravideoscope LTF-260 is indicated for use for endoscopy and endoscopic diagnosis and treatment within thoracic cavity. Our records indicate that your facility has purchased one or more of affected LTF-260.

We would herewith like to inform you that the combination of the below Endo-therapy accessories with the LTF-260 did not pass the recent compatibility assessment. If these devices are used in combination within the thoracic cavity, the efficacy (such as the amount of the tissue collected/coagulated or the amount of injection) may be lower than the users expect or minor injuries (scratches or minor bleedings) may occur inside the patient's body.

However, to date Olympus did not receive any report of injury with the LTF-260 Pleuravideoscopes used in combination with the described Endo-therapy accessories.

In an effort to maximize patient safety and mitigate any potential risk to patient health Olympus is notifying users of the incompatibility of the below mentioned Endo-therapy accessories to Pleuravideoscopes LTF-260 and requires you to **immediately stop using the LTF-260 in combination with the below listed Endo-therapy accessories.**

Type of Endo therapy	Model Name	
accessory		
Spray catheter	PW-6C-1	
Hot Biopsy Forceps	FD-6C-1,FD-7C-1	
Injector	NM-4L-1, NM-8L-1	
Electro Surgical Knife	KD-31C-1	
Coagulation electrode	CD-6C-1	


Table 1: Incompatible Endo-therapy accessories to LTF-260

OLYMPUS

The above-mentioned Endo-therapy accessories may be used with <u>other</u> Olympus endoscopes <u>if</u> the Instruction for Use of the device states the compatibility with such other endoscope (GIF, CF, BF and others)

Note: The Biopsy forceps FB-55CR-1 remains compatible to the LTF-260.

Please find below the revised system chart which you can also find in section *Endo-therapy accessories* of the updated LTF-260 Operation Manual (*GE5470 ver.15, page 65*):

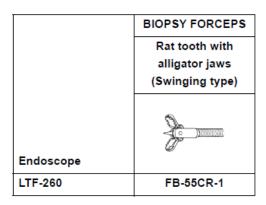

Advice on actions to be taken by the user:

Our records indicate that your facility has purchased one or more of the above-referenced endoscopes. Therefore, Olympus requires you to take the following actions:

- a) Inspect your inventory of operation manuals for the referenced Olympus models and discard the existing operation manuals.
- b) Exchange them with the updated operation manuals having updated system chart indicating compatible endoscope accessories. (Please refer to attachment 1)
- c) Notify related personnel at your institution that they should stop the combination use of the LTF-260 PLEURA VIDEOSCOPE and the subject Endo-therapy accessories (Please see Table 1).
- d) Indicate on the enclosed Reply Form that you have received and understood this Field Safety Notice.
- e) Send the completed Reply Form back to your Olympus representative.
- f) If you have further distributed this product, identify your customers, forward them this Field Safety Notice including the attachments and appropriately document your notification process.

Your national competent authority has been informed of this Field Safety Notice.

Olympus regrets any inconvenience caused and fully appreciates your prompt cooperation in addressing this situation. If you require additional information, please do not hesitate to contact Olympus directly by e-mail.

Sincerely,

Iman Ibrahim Regional Quality Assurance and Regulatory Affairs Manager Middle East & Africa Olympus MEA FZ-LLC



Registration No. 93456 (Dubai Creative Clusters Authority) Dubai Science Park - Nucleotide Complex - Dubai - United Arab Emirates P.O. Box: 33607 Dubai United Arab Emirates

Phone: +971 56 524 0079 E-Mail: iman.ibrahim@olympus-mea.com



REPLY FORM – QIL 152-008

URGENT FIELD SAFETY NOTICE NOTIFICATION OF INCOMPATIBLE ENDO-THERAPY ACCESSORIES FOR OLYMPUS EVIS LUCERA PLEURAVIDEOSCOPE LTF-260				
[Name & Address of Hospital/Medical Facility]				
[Dept/Attn]				
[Date]				
Products available in your facility				
Please insert the LTF-260 serial numbers available in your facility in the below table				
Model name	Serial number available			
LTF-260 Evis Lucera				
Pleuravideoscopes				
Additional Operation Man Should you require addition	•	a raplacament Ope	ration Manuals place	so indicato in the
table below the total quar	•	e replacement Ope	fation Manuals, pied	
Model name	Quantity of replacement manuals required			
LTF-260 Evis Lucera				
Pleuravideoscopes				

I herewith acknowledge the receipt of your Field Notice (FSN) as well as the updated Operation Manual. I understand the necessity to stop using the LTF-260 in combination with the referenced Endo-therapy accessories and confirm that I have transferred the content of the attached FSN to all affected departments on which this action has an impact.

Name (Signature)

Name (Print)	
--------------	--

Position

Please E-mail this completed reply form to Olympus at {ra@olympus-mea.com} by 26th of January, 2020.