Recall detail

Type of Product ⁱ	Medical Device	
TGA Recall Reference ⁱⁱ	RC-2015-RN-00093-1	
Product Name/Description ⁱⁱⁱ	AK 200 S and AK 200 ULTRA S nocturnal home hemodialysis machines	
	ARTG Number: 112135	
Recall Action Level ^w	Consumer	
Recall Action Classification ^v	Class I	
Recall Action Commencement Date ^{vi}	4/02/2015	
Responsible Entity ^{vii}	Gambro Pty Ltd	
Reason / Issue ^{viii}	Gambro has received complaints related to a technical error in the AK 200S noctur home haemodialysis machines, which will stop all the pumps and close the venous This has led to 3 cases of 400mL of blood loss and a 2 day treatment delay. A furth reported cases resulted in a technical error and a minor treatment delay.	clamp.
	One section in the Operator's Manuals for the AK 200 S and the AK 200 ULTRA S	
	instructs the user to contact technical service and to restart the blood pump to return	
	extracorporeal blood to the patient. Restarting the blood pump is however not poss since the venous clamp is closed and the consequence will be a blood loss for the	
	No Deaths or injury have been reported as a result of this issue.	
Recall Action ^{ix}	Recall for Product Correction	
Recall Action Instructions [×]	An addendum to the Instructions for Use is to be communicated to customers.	
Contact Information ^{xi}	02 8852 3700 - Gambro Pty Ltd	

Footnotes

ⁱ Type of Product: Medicine, Medical Device, or Biological

ⁱⁱ TGA Recall Reference: Unique number given by the TGA

iii Product Name/Description: Brand name (including active ingredient for medicines) and may include generic efference for the kind of medical devices. Includes all necessary information such as affected: catalogue / model and / or batch / serial numbers.

^{iv} Recall Action Level: The level to which the recall action is to be undertaken. This is based on the significance of the risk and the channels through which the goods have been distributed. The recall action levels are / Wholesale / Hospital / Retail / Consumer.

- Wholesale includes wholesalers and state purchasing authorities.
- Hospital includes nursing homes and institutions, hospital pharmacists, ambulance services, blood and tissue

Report generated 16/02/2015 5:08:10 PM

The TGA publishes Australian recall actions in a searchable database to ensure the public has access to information about therapeutic products that have been recalled from the Australian market. If you are concerned about your health or if you have experienced an adverse event please seek advice from a health professional as soon as possible. Please read all the important information at the beginning of this report.